


BASICS

by REDGUM

FOLDING WALKING FRAME

B4070WS

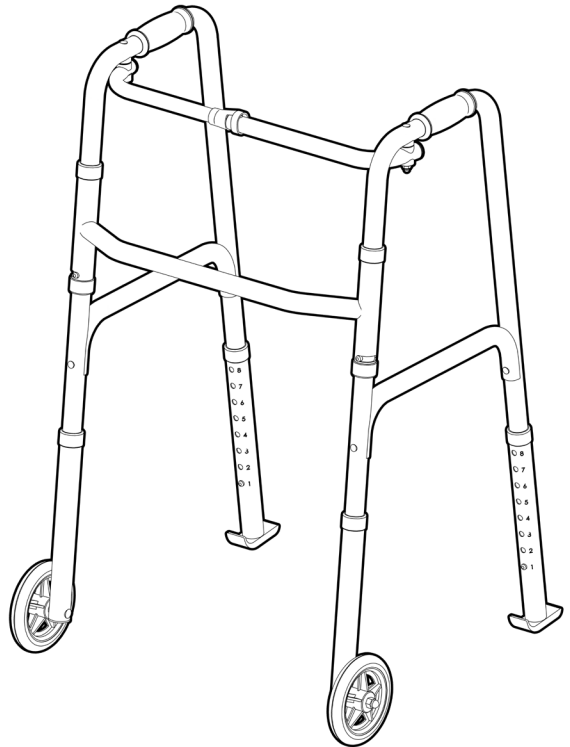

USER MANUAL

Please read and keep these instructions

BASICS

by REDGUM

BASICS by Redgum owns the **copyright** to the **Folding Walking Frame** user manual. Any information contained in this user manual shall not be photocopied, reproduced or translated into any other languages without the prior written consent of **BASICS by Redgum**.

BASICS by Redgum strives to provide the highest quality products possible. Product is subject to change without notice.

© Copyright **BASICS by Redgum** 2021




BASICS by Redgum is part of **FOR-DE Group Pty Ltd**

redgumbrand.com.au

CONTENTS

1. **INTRODUCTION** - INTENDED USE
2. **PRODUCT OVERVIEW** - FEATURES / SPECIFICATIONS
3. **SET UP** - UNPACK and INSPECT SUPPLIED PARTS
4. **SET UP** - UNFOLD FRAME
5. **ASSEMBLY** - FIT REAR GLIDES
6. **ASSEMBLY** - FIT FRONT WHEELS
7. **SET UP** - WALKING FRAME HEIGHT ADJUSTMENT
8. **USAGE** - FOLDING WALKING FRAME
9. **USAGE GUIDELINES** - DO's + DON'Ts
10. **PRODUCT CARE** - INSPECTIONS / CLEANING
11. **WARRANTY**
12. **BASICS by Redgum** - PRODUCT RANGE

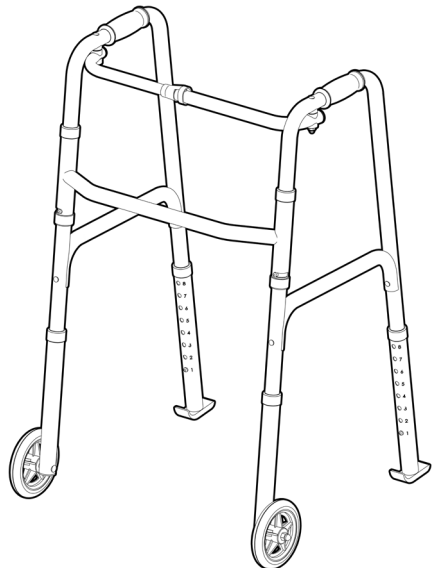

INTRODUCTION

Thank you for purchasing this quality **BASICS by Redgum** product.

If you have any issues or questions about the **Folding Walking Frame** please contact your supplier or **BASICS by Redgum** direct:

T: +61 8 9248 4180

E: sales@for-de.com.au

A: 1 Business Way, Malaga Western Australia 6090

INTENDED USE


This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances.

Only use the Folding Walking Frame for its intended purpose which is as a stability aid on which to lean on for support to maintain balance whilst standing or walking.

Refer to usage guideline page for information on how to use the Folding Walking Frame in a safe manner.

IMPORTANT NOTE


This equipment **MUST** be assembled for use by a competent person.

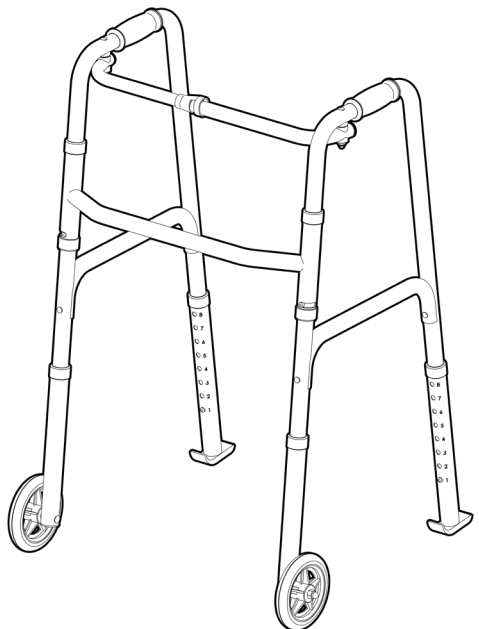

PRODUCT OVERVIEW

FEATURES

- Stable standing / walking frame
- Great independent living aid
- Adjustable height
- Ø125mm front wheels / rear glides
- Folds up for transit or storage
- Strong lightweight construction
- Comfortable hand grips

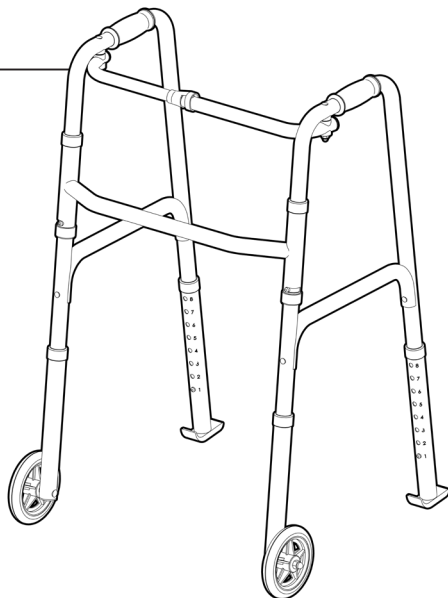

SPECIFICATIONS (All dimensions shown are nominal)

Overall:

595mm (W) - 440mm Internal (W)

530mm (D)

825 - 1000mm (H) - 8 height settings

Overall - Folded:

625mm (W)

100mm (D)

860mm (H) at lowest setting

Wheels:

Ø125 (5") x 20mm - Grey PVC tyre / Black PP rim (replacements available)

Glides:

80 x 30mm - White PP (replacements available)

Product Weight: **2.6kg**



Maximum User Weight: **136kg**

Please contact **BASICS by Redgum** if you would like any further information on the Folding Walking Frame .


SET UP - UNPACK and INSPECT SUPPLIED PARTS

Ensure you have been supplied with the following parts:

We recommend that you visually inspect the Folding Walking Frame parts prior to assembly:

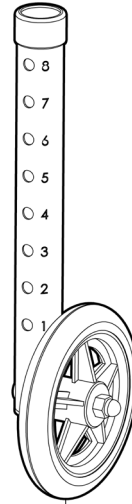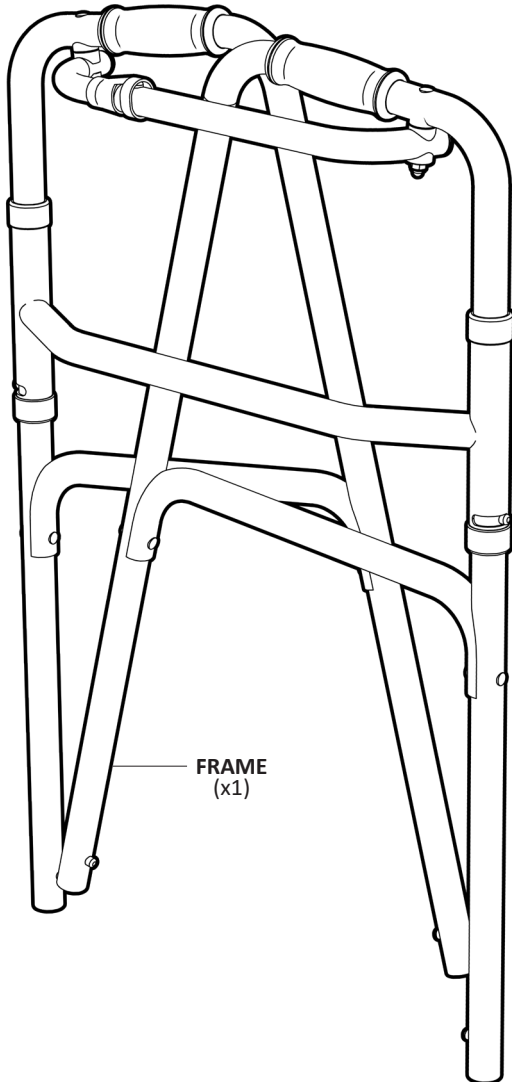

LEG WITH WHEEL
(1 x LEFT)
(1 x RIGHT)

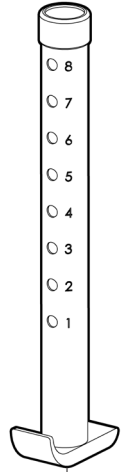

LEG WITH GLIDE
(x2)

IMPORTANT NOTE:

If you see any damage suspect a fault or are missing parts **DO NOT ASSEMBLE THE PRODUCT** and contact your supplier for support.



SET UP- UNFOLD FRAME

The Folding Walking Frame needs to be unfolded out of the box.

Please follow these steps to unfold the frame:

1. Rotate out the side frame that is on the outside of frame.
2. Rotate out the other side frame.

A loud 'click' will be heard when the side frames have been locked into position.

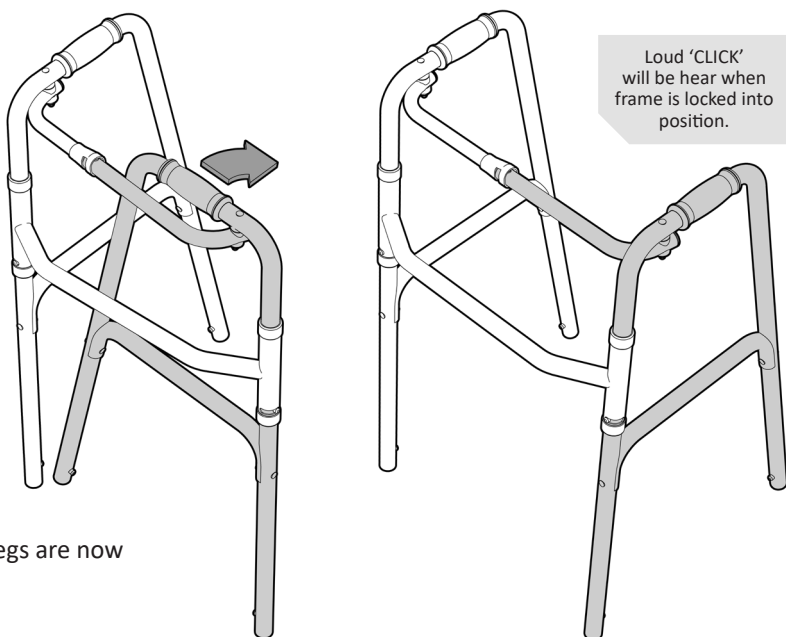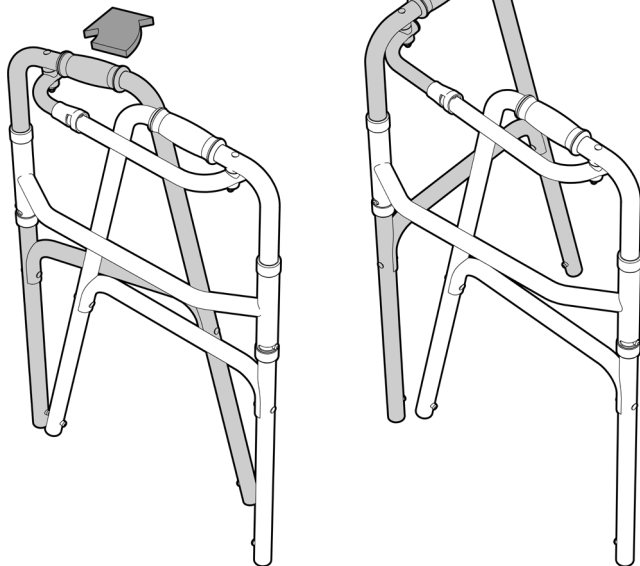


The adjustable legs are now ready to fit.

4.

ASSEMBLY - FIT REAR GLIDES

Please follow these steps to fit rear glides to frame:


1. Put the unfolded frame on its side.
2. Offer up leg with glide to REAR LEG of side frame.
Align holes in leg with press button on frame.
3. Slide leg onto frame.
4. Push in press button and continue to slide leg onto frame until press button pops out of hole in leg. (Refer to page 7 for walking frame height adjustment)
5. Repeat process to fit other leg with glide onto frame.

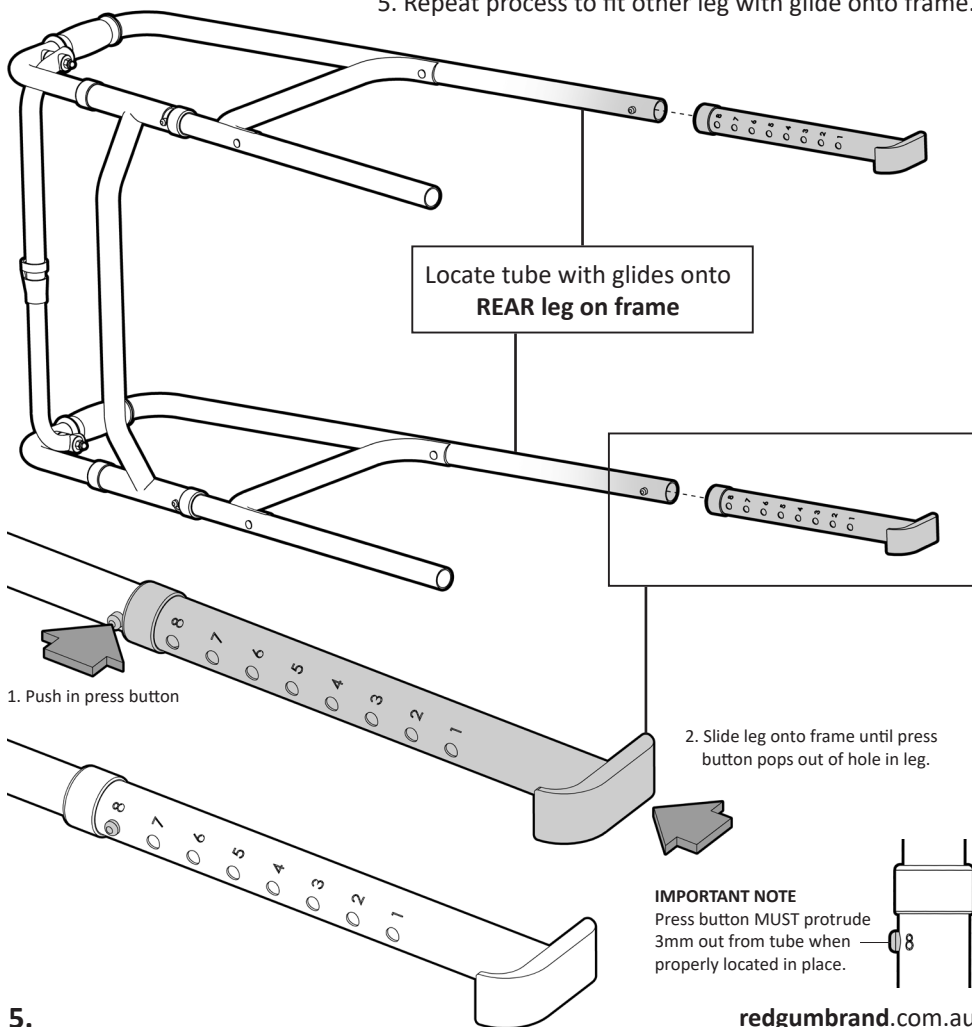

ASSEMBLY- FIT FRONT WHEELS

Please follow these steps to fit front wheels to frame:

1. Put the unfolded frame on its side.
2. Offer up leg with wheel to FRONT LEG of side frame.
Ensure wheel is facing toward OUTSIDE of frame.
Align holes in leg with press button on frame.
3. Slide leg onto frame.
4. Push in press button and continue to slide leg onto frame until press button pops out of hole in leg. (Refer to page 7 for walking frame height adjustment)
5. Repeat process to fit other leg with wheel onto frame.

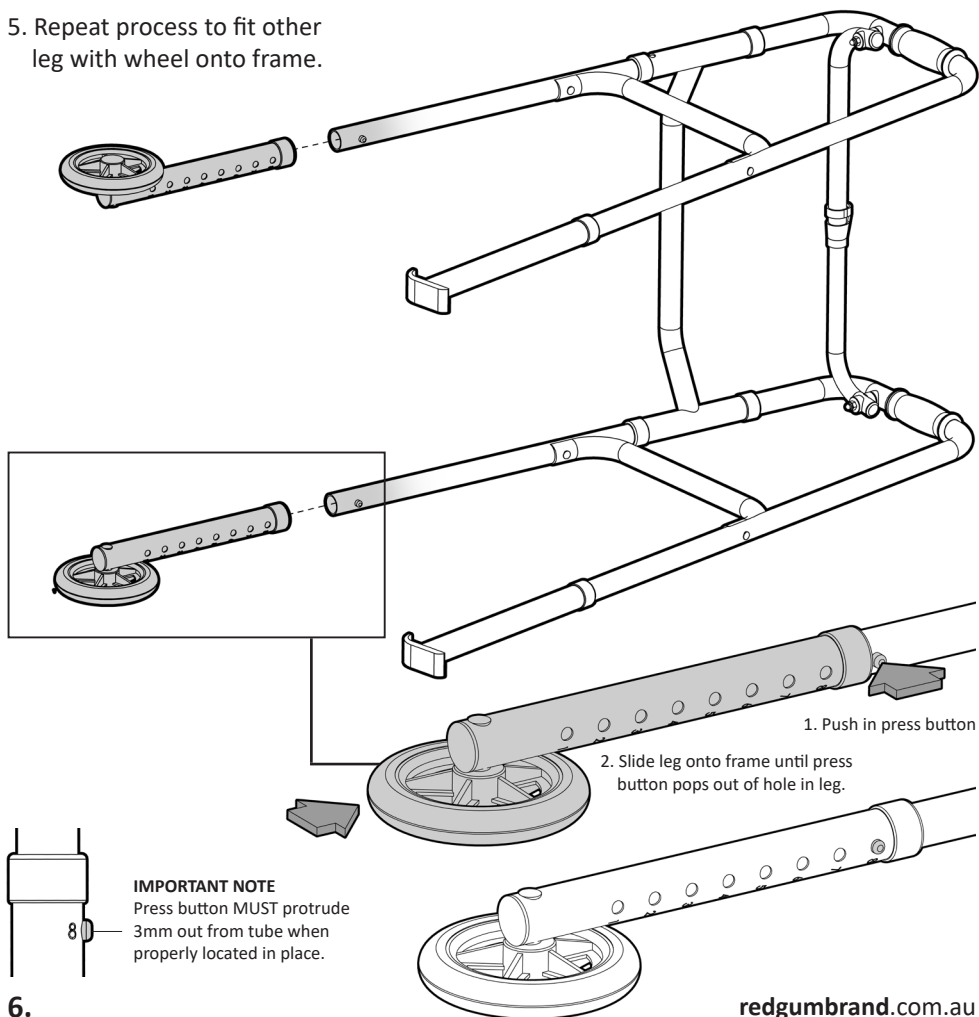

6.

SET UP - WALKING FRAME HEIGHT ADJUSTMENT


We recommend that the Folding Walking Frame be set at the correct height for the user.

The user should be able to hold onto the hand grip with a straight arm when using the walking frame.


To adjust the height of the frame:

- Push in press button and slide leg onto frame until press button pops out of required hole in leg.
- Repeat process to adjust other legs onto walking frame.

Use numbers on leg tube to ensure all legs are set at the same height.

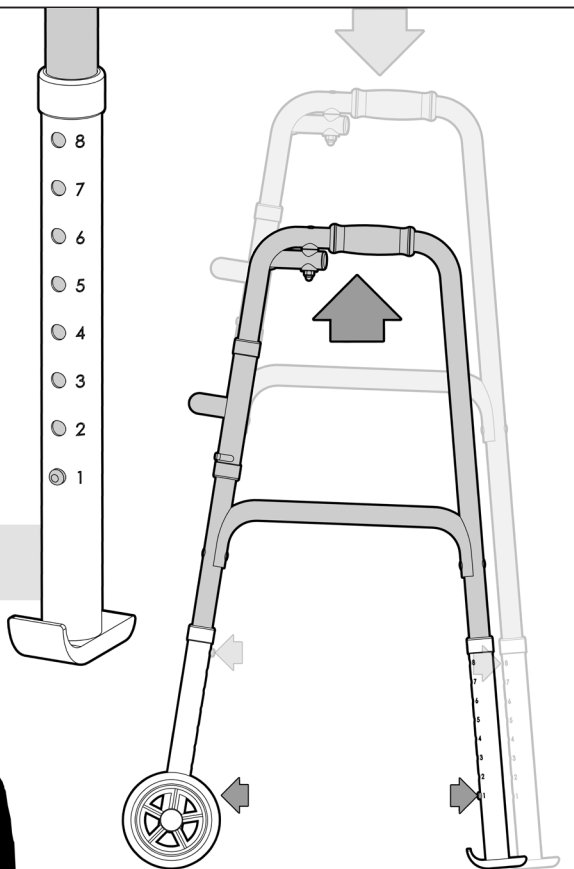


Straight arm
when holding
walking frame

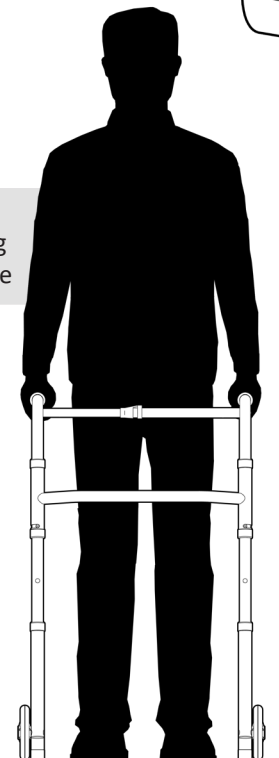

IMPORTANT NOTE

Press button MUST protrude 3mm out from tube when properly located in place.

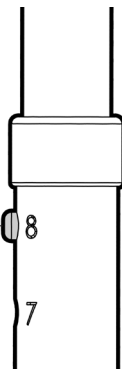


USAGE- FOLDING WALKING FRAME

The Folding Walking Frame can be folded for storage and transit.

To fold the Walking Frame follow these steps:

- Press down on the raised section of the plastic part found on the upper cross tube and rotate in the side frame.
- Rotate in the other side frame.

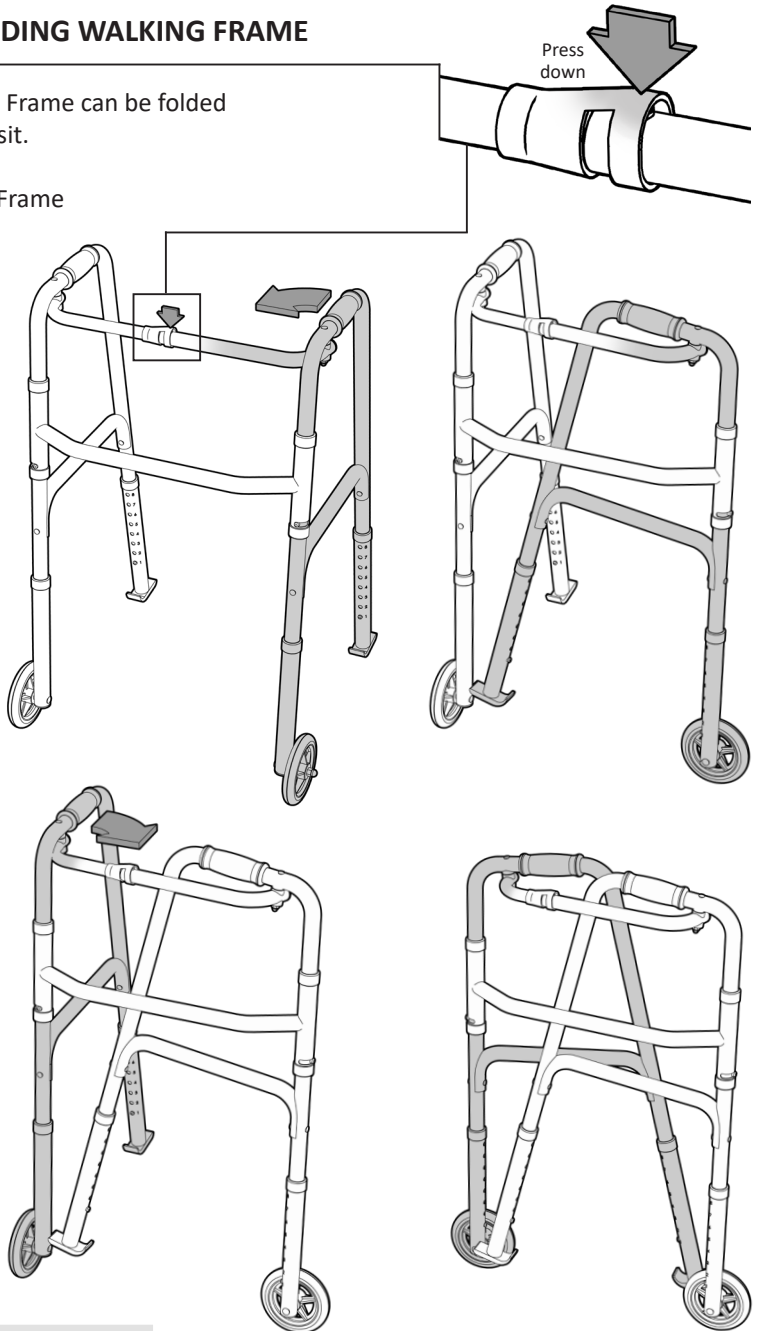

To unfold frame refer to page 4.

USAGE GUIDELINES - DO's + DON'TS

Please follow these guidelines to get the optimal benefit from the Folding Walking Frame.

Failure to follow these guidelines may result in damage to the Folding Walking Frame and put the user at risk.

The Folding Walking Frame is designed to take **136kg Maximum Load**.
Do not overload the product!

Periodically inspect to the Folding Walking Frame prior to use to ensure it is in good working order.
(Refer to page 9)

Always adjust the Folding Walking Frame to correct height for the user.
(Refer to page 7)

Never use the Folding Walking Frame to pull yourself up into a standing position as it may tip over. Use as a balancing aid once standing.



Never hang any objects (ie. bag) off the Folding Walking Frame as it might tip over.

Avoid lifting the Folding Walking Frame off the ground where possible.

Ensure wheels and glides are in contact with the ground.

Take care when using Folding Walking Frame on uneven or wet ground.

If you have any issues or questions about the Folding Walking Frame please contact your supplier or BASICS by Redgum direct.

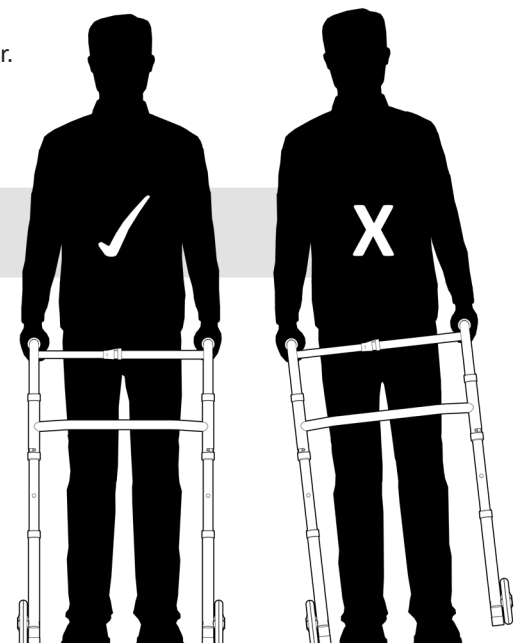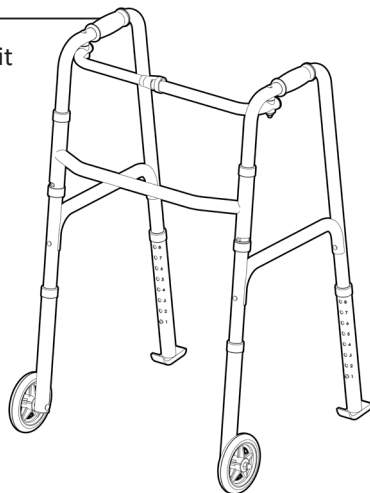

PRODUCT CARE - INSPECTIONS / CLEANING

INSPECTIONS

We suggest that the Folding Walking Frame is visually checked periodically for signs of damage or wear prior to use. Overall check the following:

- Frame has no distortion.
- Hand grips are not damaged or worn.
- Front wheels are not damaged and spin freely (Remove any hair or lint).
- Rear glides are not damaged or worn.




Withdraw the Folding Walking Frame immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CLEANING

Clean the Folding Walking Frame using a non-abrasive cleaner or mild detergent with a soft cloth.

Abrasive cleaners or abrasive cleaning pads could damage the Walking Frame and should not be used.

If you have any issues or questions about the Folding Walking Frame please contact your supplier or BASICS by Redgum direct.

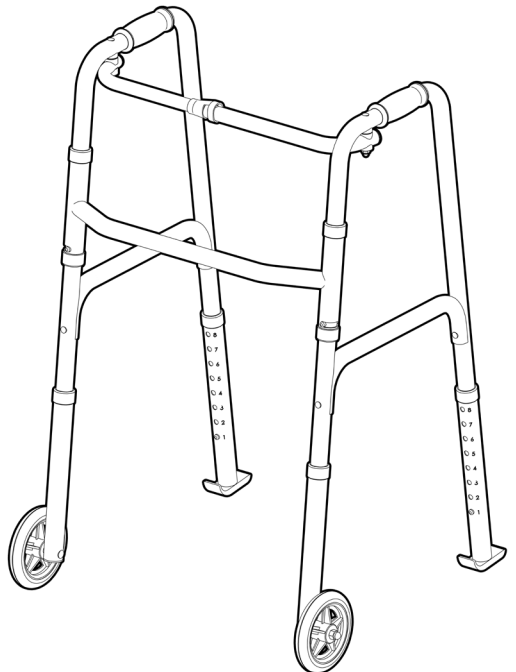

WARRANTY

The BASICS by Redgum:

B4070WS - FOLDING WALKING FRAME

is covered by a **12 Month Warranty** from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of **ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.**

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.


If you have any issues or questions about the Folding Walking Frame please contact your supplier or BASICS by Redgum direct on:

T: +61 8 9248 4180

E: sales@for-de.com.au

A: 1 Business Way, Malaga
Western Australia 6090

**Please keep this
User Manual
in a safe place
for future reference.**

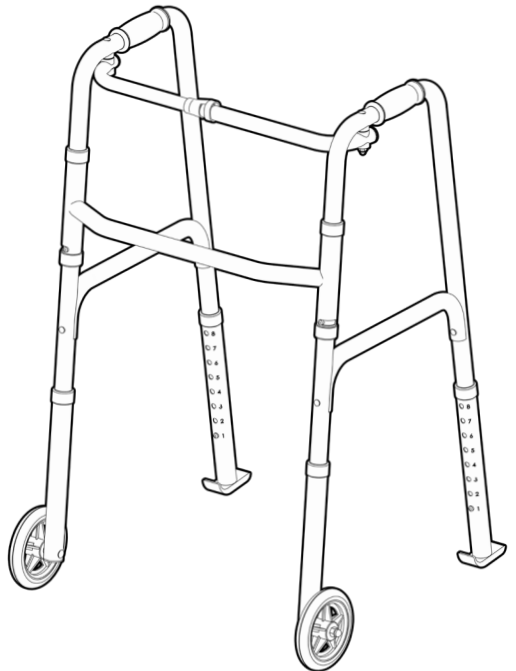


BASICS by Redum - PRODUCT RANGE


BASICS by Redgum offer a range of products including:




Step
B68H

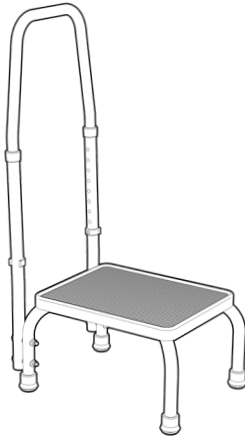

Step with handle
B68SH

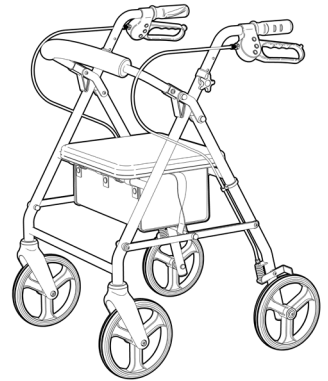

Seat walker
(Burgundy red + Silver)
B4204S

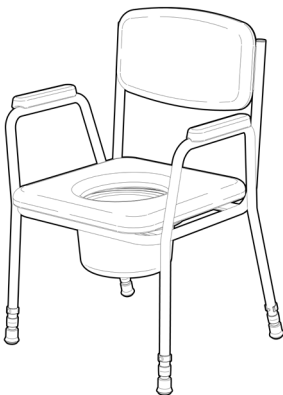

Bedside commode
B580



Bariatric commode chair
B588

Please contact BASICS by Redgum for further information:

T: +61 8 9248 4180 E: orders@for-de.com.au

A: 1 Business Way, Malaga Western Australia 6090

W: www.redgumbrand.com.au

DISTRIBUTOR:

BASICS

by **REDGUM**



Proudly Australian Owned and Operated